


The Aspen® QuikDraw PRO™ is used for pain relief.


READ INSTRUCTIONS BEFORE USE. These directions are guidelines only. For existing medical conditions, consult a physician prior to use. Severe or persistent pain may be an indication of serious medical condition. Consult a physician if such symptoms are present. Pregnant and pre and post surgery patients should use this product only as directed by a physician.

QUIKDRAW PRO™ PREPARATION

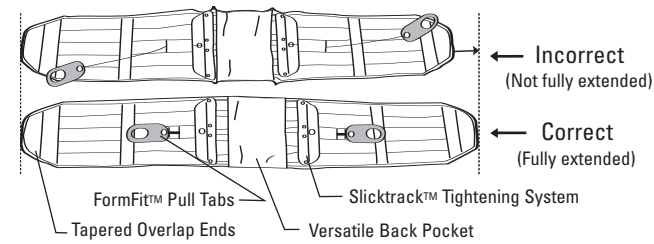


Be sure the brace is stretched to its full length before applying. The FormFit™ pull tabs must be all the way in or the brace won't stretch to its full length. When removing the brace, be sure to position the FormFit™ pull tabs as shown in the correct drawing.


PLACEMENT

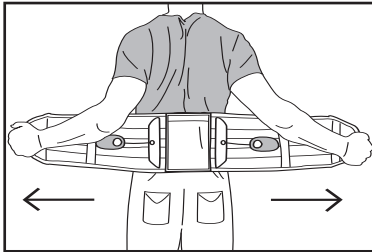


1. Apply the fully stretched brace making sure the back panel is centered and low on the back.

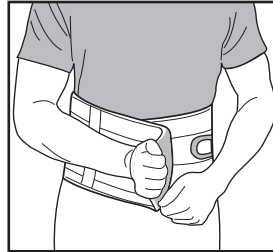


2. Overlap the tapered ends of the brace so the gray hook velcro grips the opposite side.

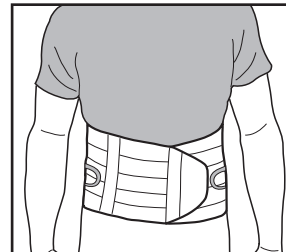


3. Inhale and apply the brace as tightly as possible.

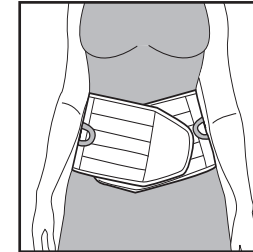

4. Note: To get a snug fit, it is often necessary to angle the two side panels as shown.

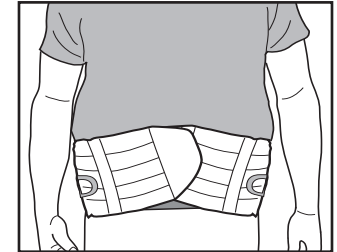


TIGHTENING

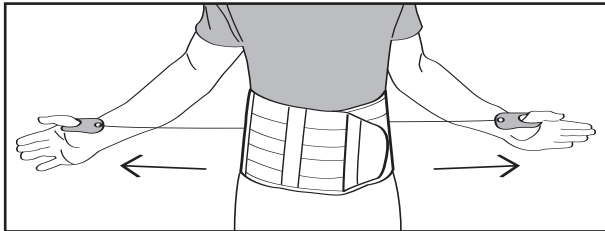


5. Slip each thumb into the FormFit™ pull tabs. Extend the tabs out and away from the brace. The left drawstring controls the tension on the lower portion of the brace, while the right drawstring controls the tension on the upper portion of the brace. Adjust and tighten the brace by increasing or decreasing the pull force on the drawstring.


ADJUSTMENT

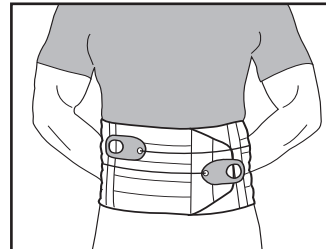


6. To secure the FormFit™ pull tabs, firmly attach the hook velcro side of the Pull tab to the loop velcro on the brace. Further adjustments can be made by releasing the velcro and tightening or loosening the drawstrings (and then lifting grips) and reapplying to desired tension. Note: For sitting, it may be desirable to loosen the drawstrings, which loosen the upper and lower portions of the brace.

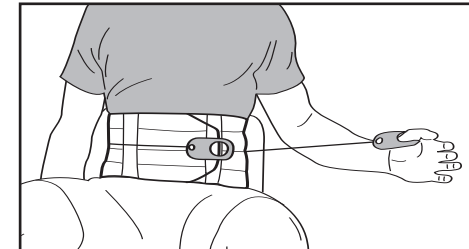

BACK POCKET FEATURES

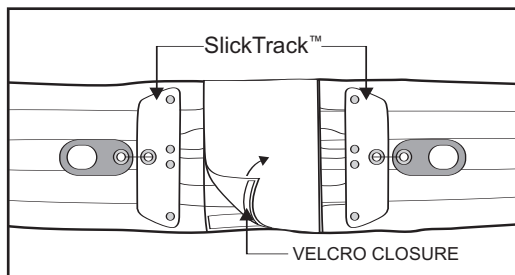

1. To access back pocket features, open the back flap located between SlickTrack™ Tightening System.

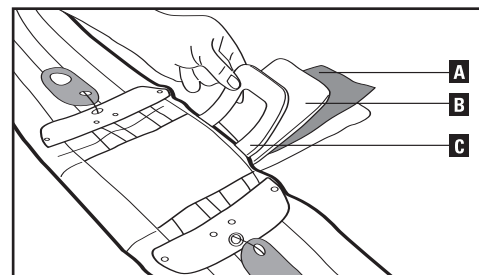

2. The back pocket contains the rigid posterior panel and a foam pad. There is also space for an optional cold therapy pack.

- A** The option of a cold pack is also available for additional therapy. The cold pack (sold separately) is in the A position within the back pocket.
- B** The cushion foam pad provides additional comfort as a barrier between the low back and the rigid posterior panel. This foam pad is in the B position within the back pocket.
- C** The customizable rigid posterior panel can mimic the lordotic curve of the low back. This panel is in the C position within the back pocket.



The Aspen QuikDraw PRO™ sizing follows closely to standard pants sizing. Size ranges are only guidelines. Your experiences and personal assessment could make the difference in choosing the correct size brace.

QUIKDRAW Pain Relief Orthosis		Men's Pant Size	Women's Pant Size
Extra Small	21 to 27 in. (46 to 61 cm)	22 to 26 in.	Child - 0
Small	26 to 32 in. (68 to 81 cm)	26 to 30 in.	2 - 4
Medium	31 to 37 in. (79 to 94 cm)	30 to 34 in.	6 - 10
Large	36 to 42 in. (91 to 107 cm)	34 to 38 in.	12 - 16
Extra Large	41 to 47 in. (104 to 119 cm)	38 to 44 in.	18 - 22
Extra Extra Large	46 to 57 in. (117 to 145 cm)	44 to 54 in.	24 - 34



Aspen Medical Products, Inc.
6481 Oak Canyon, Irvine, CA 92618
800-295-2776 • 949-681-0200
www.aspenmp.com



ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.

LIMITED WARRANTY

Aspen Medical Products, Irvine, CA, 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or part(s).

Aspen Medical Products shall have no obligation under this limited warranty in the event:

- (a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;
- (b) The product is altered;
- (c) Any parts not supplied by Aspen Medical Products are inserted into the product; or
- (d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.


This warranty gives you specific legal rights and you may have other rights which vary from state to state.

Warning: To ensure proper use, please review all material.

A listing of authorized distributors in your area is available upon request.



6481 Oak Canyon
Irvine, CA 92618
949-681-0200
800-295-2776
www.aspenmp.com

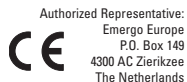

© 2005 Aspen Medical Products, Inc. All Rights Reserved.