medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Austria GmbH
Adamgasse 16/7
A-6020 Innsbruck / Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at

medi Belgium bvba
Voortstraat 79
3580 Beringen / Belgium
T +32 11 24 25 60
F +32 11 24 25 64
info@medibelgium.be
www.medi Belgium.be

medi Danmark
Vejlegardsvej 59
2665 Vallensbæk Strand /
Denmark
T +45 46 55 75 69
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona / Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi UK Ltd.
Plough Lane
Hereford HR4 OEL / United
Kingdom
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

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Shoulder immobilisation support

Indications

- Conservative and postoperative treatment following proximal fractures of the head of the humerus
- Conservative and postoperative treatment of fractures of the scapula
- Conservative and postoperative treatment of AC joint dislocation
- Conservative therapy of painful osteoarthritis of the shoulder
- Conservative and postoperative treatment of impingement syndrome
- Conservative and postoperative treatment of osteoarthritis of the AC joint
- Postoperative treatment after shoulder replacement
- Conservative treatment after dislocation of the shoulder
- Conservative treatment of chronic dislocation of the shoulder
- Conservative and postoperative treatment of fractures of the distal humerus

Contraindications

None known at present.

Fitting instructions

1. Wrap the wider strap around your body and fasten it just below the chest.
2. Fix the injured arm with the strap around the upper arm. Attach the lower arm sling to the abdominal strap. Open the sling and place the injured arm inside. Close the sling afterwards.
3. Pass the thin strap over the healthy shoulder and attach it to the lower arm sling. Adjust the length of the strap to the required length.

Care instructions

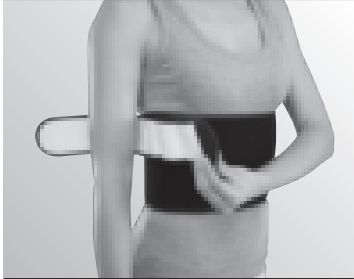
The support can be washed by hand in mild, cool soapy water. Leave to air dry. Soap residues can cause skin irritation and damage to the material.

Storage instructions

Please keep the brace in a dry place and do not expose it to direct sunlight.

Your medi team

wishes you a quick return to full fitness.

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